## assembly instructions

Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.

1. First remove all contents from carton.
2. Open frame and slide padded backrest into frame brackets making sure snap bottoms protrude through holes.
3. Remove packaging caps on handle receptacle and insert handle tube into frame. The brake cable should be positioned on outside of frame. Set handle to desired height. Insert bolt into the clamp from the inside of the frame. Next slide the washer on the bolt from the outside of the frame. Next attach tightening knob to the bolt and turn clockwise until secure. Make sure both handles are set to the same height before use.

**WEIGHT CAPACITY 300 LBS (136 KG)**

## safety precautions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from rolling. If the wheels roll while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should roll freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

### Important Safety Notice

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

## loop lock operation

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop—Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position—Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks—Pull up on both loop locks with palms of hand to original position.

## instructions to adjust the hand brake system


### Important Safety Notice

The hand brake system fitted on this rallator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

Hold firmly in place do not turn.

To tighten the hand brake adjustment. "turn the nut clockwise"

To loosen the hand brake adjustment. "turn the nut counter clockwise"

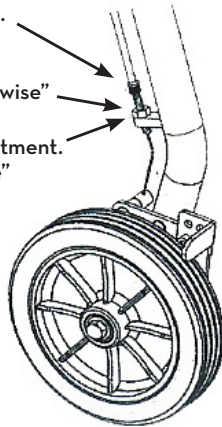## maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit [www.drivemedical.com/help](http://www.drivemedical.com/help) for more info.

## lifetime limited warranty

Your SkyNet brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

**⚠ WARNING:** This product can expose you to chemicals, including di (2-ethylhexyl) phthalate (DEHP), which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

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